



For Dogs and Cats.

**Caution:** This product is restricted to use by or on the order of a veterinarian.

### **Oculenis™ BioHAnce™ Ocular Repair**

**Gel** is formulated to support enhanced healing of acute injuries, surgical incisions and common irritations in the eyes of dogs and cats. BioHAnce is a unique, crosslinked hyaluronic acid (HA) that has been specifically modified for the ocular environment. It creates a cellular matrix of modified HA that naturally enhances the animal's own healing process by optimizing epithelial proliferation. It also provides a sheer coating on the surface of the eye that protects the affected area and helps reduce scarring.

### **Indications**

For use on superficial ocular injuries (cuts, abrasions, scratches), traumatic ocular wounds (lacerations, punctures, foreign objects that have perforated the cornea); surgical incisions; and corneal ulcers.

### **Effectiveness**

The efficacy of this product is supported with research conducted by a major university in the United States. The patented BioHAnce modified hyaluronic acid crosslinking technology has been shown to help accelerate the rate of healing in acute ocular injuries.

### **Ingredients**

All ingredients are naturally occurring in the body and do not cause pain or stinging when applied.

- Water
- BioHAnce Hyaluronic Acid (0.75%)
- BioSalts
  - Sodium Chloride
  - Disodium Phosphate
  - Potassium Chloride
  - Potassium Phosphate

Contains no preservatives or excipients that may irritate the eyes. Free of antibiotics or steroids.

**Warning:** For animal use only.

### **Adverse Reactions**

For product support or to report an adverse reaction call 833-224-2015.

### **Administration**

1. Place 1 to 2 drops directly onto the injured eye. The natural movement of the eyelid will distribute the gel over the eye surface.
2. Oculenis BioHAnce Ocular Repair Gel should be applied a minimum of 2 times daily until eye injury is repaired per veterinarian consultation.
3. To avoid contamination, do not touch the top of the container to any surface.
4. Replace cap securely after using.

### **Storage**

Product should be stored at room temperature, not above 85°F. May be stored in the refrigerator. If refrigerated, the product should be allowed to warm to room temperature prior to use. DO NOT FREEZE.

### **How Supplied**

3 mL dropper bottle

For customer service, contact SentrX™ at 833-224-2015.

Manufactured by  
SentrX Animal Care, Inc.  
Salt Lake City, UT 84108

